



Mixing Table

CALMATRIX® Calcium Sulfate Bone Graft Binder

BONE GRAFT PRODUCT	.17g of CALMATRIX	.34g of CALMATRIX
DBM (Autograft or Allograft)	.25cc	.5cc
DBM (Autograft with binder)	.5cc	1.0cc
Synthetic Bone (Alloplast – 250-1000µm)	.25cc	.5cc
Marine/Animal (Xenograft)	.25cc	.5cc

INDICATIONS FOR USE:

- Intrabony/infrabony defects
- Furcation defects
- Recession defects
- Dehiscence/fenestration defects
- Extraction socket (ridge preservation) defects (natural teeth and prosthetic root form implants)
- Ridge augmentation
- Sinus lifts
- Endodontic bony defects



Manufacturer

Lifecore Biomedical, Inc.
3515 Lyman Blvd.
Chaska, MN 55318 USA
Call: 800-752-2663
Fax: 800-651-8521

Outside the USA

Call: +1-952-368-4300
Fax: +1-952-368-4324

Authorized Representative

Lifecore Biomedical, SpA
via A. Fleming, 19
37135 Verona, Italy
Call: +39-045-8230294
Fax: +39-045-8250296

Subsidiary:

Lifecore Biomedical, GmbH
Alfter, Germany
Call: +49-2222-9294-0
Fax: +49-2222-977356

Subsidiary:

Lifecore Biomedical, S.A.S.
Beauzelle, France
Call: +33-5-62-21-40-45
Fax: +33-5-61-42-06-32

Subsidiary:

Lifecore Biomedical, AB
Särö, Sweden
Call: +46-31-93-68-23
Fax: +46-31-93-68-45