

THE SHAPE IS UP TO YOU

*Innovative and unique solutions
to your surgical needs*

CALFORMA[®] Calcium Sulfate Bone Graft Barrier

Protecting your bone graft has never been this SIMPLE...


CalForma is a shapeable putty-like containment barrier that covers intra-oral defects and provides protection and undisturbed space for regeneration. CalForma prevents invasion of undesirable tissue, cells, and bacteria and prevents scattering of bone graft material.

The formulation includes:

- Calcium Sulfate Alpha-hemihydrate
- Hydroxypropylmethylcellulose (HPMC)
- Hydrating Solution

CALFORMA Clinical Advantages:


- Provides barrier protection over numerous defects.
- Maintains handling characteristics in the presence of blood and saliva.
- Resorbs in approximately 4-6 weeks into its component parts naturally found in the body.
- Soft tissues migrate over CalForma 1-3 weeks where primary closure is not achieved.
- Saves valuable chair time.

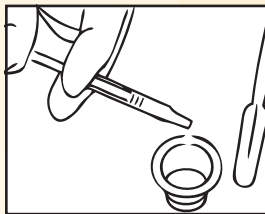


Insert and screw plunger rod into syringe. Add entire hydrating solution. Mix vigorously and thoroughly.



Remove CALFORMA mixture. Manipulate to the approximate contour.

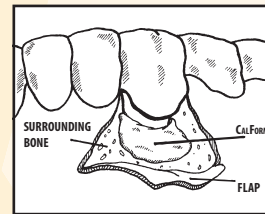

Place CALFORMA immediately. Smooth edges with the spatula quickly and lightly.

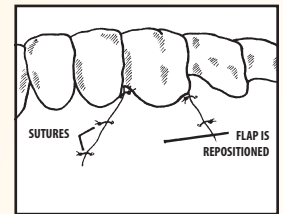

Close surgical flaps. CALFORMA sets hard in less than 4 minutes

For additional information
call your local product specialist
or call us toll free at:

1-877-243-6724
www.citagenix.com

