

UNIQUELY FLEXIBLE... CLINICALLY PREDICTABLE!

*Innovative and unique solutions
to your surgical needs*

CALMATRIX® Calcium Sulfate Bone Graft Binder

Placing your bone graft is now **EASIER** than ever...


Combined with your preferred particulate bone graft material, CalMatrix forms a putty-like composite that fills defects to predictably regenerate bone. CalMatrix is easily molded to conform to any osseous or periodontal defect.

The formulation includes:


- Calcium Sulfate Alpha-hemihydrate
- Sodium Carboxymethylcellulose
- Hydrating Solution

CALMATRIX Clinical Advantages:

- Reduces migration of bone graft particles.
- Resorption corresponds to the natural rate of bone formation, creating space for vital bone.
- Creates a stable structure in the presence of fluids.
- Uniform particle size, form, and structure offer the clinician predictable results.

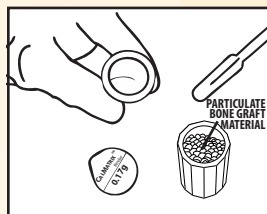


Mix CALMATRIX with particulate bone graft material of your choice.

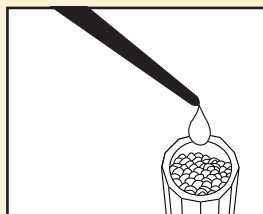

Add the syringe liquid provided in the CALMATRIX package and mix with spatula.



Form composite bone graft putty into desired shape.

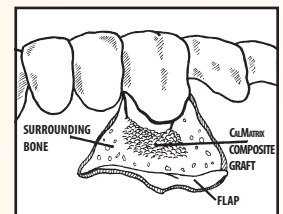

Place composite bone graft into the defect, layering it or inserting as a mass.

For additional information
call your local product specialist
or call us toll free at:

1-877-243-6724

www.citagenix.com

